



RCT Kit for ClO₂

Raven Culture Test Kit for Chlorine Dioxide Sterilization

Raven Labs now offers a biological indicator culture test kit specifically for use with Chlorine Dioxide sterilization. The RCT Kit for ClO₂ consists of 25 spore strips and 25 tubes of culture media conveniently packaged together.



Catalog # C-1-CD

- Spore Strips in Tyvek/Tyvek® pouches
- Vivid Culture Media color change
- Resistance determined for ClO₂
- Convenient packaging

The spore strips provided are packaged in Tyvek/Tyvek® pouches so that each side is permeable to the sterilant. Lot number and expiry are printed on each pouch. Each lot's specific resistance to Chlorine Dioxide is documented on the Certificate of Analysis accompanying each order.

The culture media contains Bromocresol Purple pH indicator which greatly increases the ease of use by utilizing Raven's vivid purple to yellow color change

Aseptically transfer the spore strip into the culture media provided and incubate to verify Chlorine Dioxide sterilization.

